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NOTARIAL CERTIFICATE

LAW FIRM IDEA & NOTARY OFFICE INC.

(Cheongjin-dong, Samsong Bldg.) 701, 702,
34, Jong-ro 3-gil, Jongno-gu, Seoul, Republic of Korea

10.12.2024

«Approved»

(signature)

Kim Beom Jun,

CEO

Dental Studio Co. Ltd.

**INSTRUCTIONS FOR USE
FOR THE MEDICAL DEVICE**

Karl Fritz® medical dental pincettes

1. Device name

Karl Fritz® medical dental pincettes

I. Design versions, 1 pc/pack:

1. College anatomical pincettes, P/N 5301;
2. College anatomical pincettes with lock, P/N 5302;
3. College anatomical wide pincettes, P/N 5303;
4. Pincettes for articulation paper, P/N 5306;
5. Adson anatomical straight pincettes, P/N 5351;
6. Adson-Brown surgical straight pincettes, 12 cm, P/N 5352;
7. De Bakey surgical double curved pincettes, 15 cm, P/N 5353;
8. Semken-Taylor surgical pincettes, 1x2 curved, 12.5 cm, P/N 5354;
9. Adson surgical pincettes, 1x2 straight, 15 cm, P/N 5355;
10. Adson surgical pincettes, 1x2 straight, 12 cm, P/N 5356;
11. Semken-Taylor surgical pincettes, 1x2 straight, 14 cm, P/N 5357;
12. MICRO microsurgical straight pincettes, 11.5 cm, P/N 5358;
13. MICRO microsurgical curved pincettes, 11.5 cm, P/N 5359;
14. Surgical pincettes, 18 cm, P/N 5361;
15. EASY surgical pincettes, 1x2 straight, 15.5 cm, P/N 5362;
16. EASY surgical pincettes, 1x2 straight, 17.5 cm, P/N 5363;
17. EASY surgical pincettes, 1x2 curved, 12.5 cm, P/N 5364;
18. Pincettes for tunnel technique, P/N 7101.

II. Instructions for use – 1 pc./pack. (in paper and electronic form)

Hereinafter — instruments, devices, medical device, pincettes.

2. Information about the manufacturer, developer and authorized representative of the manufacturer

Manufacturer / Developer:

Dental Studio Co., Ltd.

B101, B102, B125, B1, Geumjeong Station 2nd SKV1 Tower, 136 LS-ro, Dongan-gu, Anyang-si, Gyeonggi-do, Republic of Korea

Tel.: +82-31-8069-7007

Fax: +82-31-8069-7009

E-mail: bjkim@dentalstudio.co.kr

Web-site: www.dentalstudio.co.kr

Manufacturer's authorized representative in the Russian Federation

DENTALSTOM LLC

143432, Moscow region, Krasnogorsk, Nakhabino work settlement, Chkalova Str. 7, space LXXV

Tel./fax: +7 (495) 118-76-40

E-mail: info@dn-stom.ru

Web-site: www.dn-stom.ru

3. Intended use of the device/Indications.

The pincettes are designed to grasp and hold tissues, auxiliary and diagnostic materials and accessories.

4. Potential consumers.

Dental surgeons, maxillofacial surgeons and implantology surgeons.

5. Scope of application, contraindications and possible side effects

Scope of application: dentistry, maxillofacial surgery, dental surgery.

Contraindications: possible allergic reaction to materials.

Side effects: Not known.

6. Precautions.

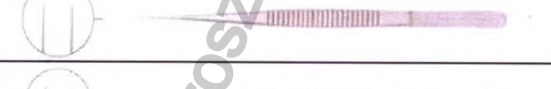
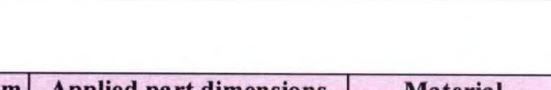
1. The instruments are supplied in a non-sterile package. It is necessary to sterilize the instruments before using them.
2. Check the instruments for mechanical damage. Before using each instrument, the specialist must make sure that the instruments are in working condition; their surfaces shall be free of any contaminations.
3. The instruments must be used in accordance with their intended purpose and instructions for use.
4. This device can be used only by dentists who received qualifications in the application of appropriate treatment methods and completed training courses.

5. Surgical interventions should be carried out in sterile conditions, medical staff should use masks, gloves, sterile clothes.

7. Description, design of devices and application procedure.

The pincettes are designed to grasp and hold tissues, auxiliary and diagnostic materials and accessories. These are reusable products.

Description: pincettes is intended to capture and hold tissues, auxiliary and diagnostic materials and accessories. The action of the pincette is based on the convergence of rods and the retention of items with jaws.

Product name	Description	Appearance
College anatomical pincettes, P/N 5301	Curved notched jaws.	
College anatomical pincettes with lock, P/N 5302	Curved notched jaws.	
College anatomical wide pincettes, P/N 5303	Curved notched jaws.	
Pincettes for articulation paper, P/N 5306	Straight unnotched jaws.	
Adson anatomical straight pincettes, P/N 5351	Straight unnotched jaws.	
Adson-Brown surgical straight pincettes 12 cm, P/N 5352	Straight unnotched jaws.	
De Bakey surgical double curved pincettes 15 cm, P/N 5353	Curved jaws with double row of notches	
Semken-Taylor surgical pincettes, 1x2 curved 12.5 cm, P/N 5354	Toothed jaws 1x2, curved.	
Adson surgical pincettes, 1x2 straight 15 cm, P/N 5355	Toothed jaws 1x2, straight.	
Adson surgical pincettes, 1x2 straight 12 cm, P/N 5356	Toothed jaws 1x2, straight.	
Semken-Taylor surgical pincettes, 1x2 straight 14 cm, P/N 5357	Toothed jaws 1x2, straight.	
MICRO microsurgical straight pincettes 11.5 cm, P/N 5358	Straight unnotched jaws.	
MICRO microsurgical curved pincettes 11.5 cm, P/N 5359	Unnotched jaws, curved.	
Surgical pincettes 18 cm, P/N 5361	Straight notched jaws.	
EASY surgical pincettes, 1x2 straight 15.5 cm, P/N 5362	Toothed jaws 1x2, straight.	
EASY surgical pincettes, 1x2 straight 17.5 cm, P/N 5363	Toothed jaws 1x2, straight.	
EASY surgical pincettes, 1x2 curved 12.5 cm, P/N 5364	Toothed jaws 1x2, curved.	
Pincettes for tunnel technique, P/N 7101	Notched jaws with open lug, curved.	

Technical specifications

Product name	Length, mm	Applied part dimensions	Material
--------------	------------	-------------------------	----------

		(L / W), mm	
College anatomical pincettes, P/N 5301	154.0	17.5 / 12.5	Stainless steel
College anatomical pincettes with lock, P/N 5302	152.0	17.5 / 12.0	Stainless steel
College anatomical wide pincettes, P/N 5303	155.0	17.5 / 12.5	Stainless steel
Pincettes for articulation paper, P/N 5306	155.0	55.0 / 35.0	Stainless steel
De Bakey surgical double curved pincettes 15 cm, P/N 5353	150.0	15.0 / 2.0	Stainless steel
Semken-Taylor surgical pincettes, 1x2 curved 12.5 cm, P/N 5354	125.0	8.0 / 1.5	Stainless steel
EASY surgical pincettes, 1x2 curved 12.5 cm, P/N 5364	125.0	8.0 / 1.0	Stainless steel
Pincettes for tunnel technique, P/N 7101	158.0	10.0 / 3.5	Stainless steel
Product name	Length, mm	Applied part width, mm	Material
Adson anatomical straight pincettes, P/N 5351	110.0	2.0	Stainless steel
Adson-Brown surgical straight pincettes 12 cm, P/N 5352	120.0	2.0	Stainless steel
Adson surgical pincettes, 1x2 straight 15 cm, P/N 5355	150.0	2.0	Stainless steel
Adson surgical pincettes, 1x2 straight 12 cm, P/N 5356	120.0	1.8	Stainless steel
Semken-Taylor surgical pincettes, 1x2 straight 14 cm, P/N 5357	140.0	2.0	Stainless steel
MICRO microsurgical curved pincettes 11.5 cm, P/N 5359	115.0	0.8	Stainless steel
Surgical pincettes 18 cm, P/N 5361	180.0	2.0	Stainless steel
EASY surgical pincettes, 1x2 straight 15.5 cm, P/N 5362	155.0	0.7	Stainless steel
EASY surgical pincettes, 1x2 straight 17.5 cm, P/N 5363	175.0	1.2	Stainless steel
Product name	Length, mm	Applied part length, mm	Material
MICRO microsurgical straight pincettes 11.5 cm, P/N 5358	115.0	35.0	Stainless steel

The procedure for using pincettes:

1. Clean, disinfect and sterilize the device;
2. Select an instrument that is suitable in size and purpose;
3. Bring the handles together and hold soft tissue or dental materials.

8. Package contents

Karl Fritz® medical dental pincettes

- I. Design versions, 1 pc/pack:
 1. College anatomical pincettes, P/N 5301;
 2. College anatomical pincettes with lock, P/N 5302;
 3. College anatomical wide pincettes, P/N 5303;
 4. Pincettes for articulation paper, P/N 5306;
 5. Adson anatomical straight pincettes, P/N 5351;
 6. Adson-Brown surgical straight pincettes, 12 cm, P/N 5352;
 7. De Bakey surgical double curved pincettes, 15 cm, P/N 5353;
 8. Semken-Taylor surgical pincettes, 1x2 curved, 12.5 cm, P/N 5354;
 9. Adson surgical pincettes, 1x2 straight, 15 cm, P/N 5355;
 10. Adson surgical pincettes, 1x2 straight, 12 cm, P/N 5356;
 11. Semken-Taylor surgical pincettes, 1x2 straight, 14 cm, P/N 5357;
 12. MICRO microsurgical straight pincettes, 11.5 cm, P/N 5358;
 13. MICRO microsurgical curved pincettes, 11.5 cm, P/N 5359;
 14. Surgical pincettes, 18 cm, P/N 5361;
 15. EASY surgical pincettes, 1x2 straight, 15.5 cm, P/N 5362;
 16. EASY surgical pincettes, 1x2 straight, 17.5 cm, P/N 5363;
 17. EASY surgical pincettes, 1x2 curved, 12.5 cm, P/N 5364;
 18. Pincettes for tunnel technique, P/N 7101.
- II. Instructions for use – 1 pc./pack. (in paper and electronic form)

9. Decoding of symbols used in packaging labeling

Symbol	Meaning	Symbol	Meaning
	Catalogue number		Caution
	Lot code		Read instructions for use or see electronic instructions for use
	Date of manufacture		
	Use before		QR-code
	Manufacturer		Non-sterile

10. Conditions of storage, transportation and operation

Storage conditions:

- store in a dry place protected from direct sunlight;
- temperature: +1 to +40 °C;
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

Transportation conditions:

Transportation of devices packaged into a transport container shall be carried out by all types of covered vehicles in compliance with the precautions indicated on the transport container and in accordance with the rules of cargo transportation applicable to that type of transport.

- temperature: -50 to +50 °C;
- relative humidity: 0-90 %;
- atmospheric pressure: 800-1 060 hPa.

Indoor conditions during medical device use

- temperature: +1 to +40 °C
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

11. Cleaning, disinfection and sterilization of devices

Used instruments shall be cleaned, disinfected and sterilized before being sent for storage.

1. Pre-sterilization cleaning

Instruments shall be cleaned and disinfected before sterilization.

- 1) Immerse the instruments with the remaining impurities (blood, soft tissues, bone fragments, etc.) in water. The instruments shall be kept in water before the following cleaning steps are started manually or in an ultrasonic bath.

2-1) Manual cleaning and disinfection of the instruments

a. Immerse the instruments with the remaining impurities in an enzyme disinfectant detergent diluted in water (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation).

b. Cleaning of all visible contaminations from the surface of surgical instruments is performed using a soft nylon brush with a short pile (cleaning time of each instrument is at least 4 minutes). Cleaning of the delicate applied parts of the instruments is carried out using cleaning needles.

c. The instruments are rinsed under running water to remove any remaining disinfectant.

2-2) Cleaning and disinfection of instruments in an ultrasonic bath (recommended by the manufacturer as the preferred method)

a. The instruments shall be completely immersed in the cleaning solution. It is recommended to use an enzyme disinfectant cleaner (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation).

b. The instruments are to be cleaned in an ultrasonic bath for 10-20 minutes, then rinsed with water.

c. The instruments are to be rinsed under running water to remove any remaining cleaning disinfectant.

- 3) Repeat steps a) and b) until the instruments are completely cleaned, then perform visual inspection of the instruments.

4) The instruments are thoroughly dried naturally or using drying cabinets for surgical instruments. If necessary, manual drying using napkins made of non-abrasive lint-free fabric is possible.

5) For more information and data on using the available enzyme and non-enzyme disinfectants, refer to the instructions of the manufacturers of these products.

2. Steam sterilization

After cleaning and disinfecting the instruments, it is necessary to carry out sterilization.

The instruments shall be sterilized by steam under pressure in an autoclave at a temperature from 132 °C to 134 °C for 15 minutes, followed by drying for 30 minutes.

12. Disposal

Medical devices shall be disposed of in accordance with the medical procedures adopted by the medical institutions, and the local, state, or federal regulations. The choice of methods for the safe disposal and destruction of devices depends on the capacity and profile of the medical organization, the availability of waste disinfection/neutralization facilities, the method of neutralization/destruction of waste adopted in the administrative territory (incineration, removal to landfills, disposal).

Instruments are considered epidemiologically hazardous waste (devices contaminated with blood and/or other biological fluids of patients, as well as devices in contact with patients with infectious diseases). In this case, a medical organization only collects and temporarily stores the above-specified waste. Collection is carried out into single-use bags or containers with color labeling adopted for hazardous medical waste in a room or in places for temporary storage of medical waste before subsequent transportation to the place of destruction or disposal. Instruments in contact with the patient shall be pre-cleaned and disinfected in accordance with the established procedure before being disposed of. Transportation of medical waste from medical institutions, and their disposal is carried out by specialized organizations having a license to handle epidemiologically hazardous medical waste.

Consumer package and unused defective devices belong to epidemiologically safe waste, similar in composition to solid household waste (SHW).

13. Shelf life

The shelf life of the device is unlimited.

14. Service life of devices

With proper use, the service life of the instruments is unlimited, except for the effects of natural wear.

15. Maintenance and repair

Maintenance and repairs are not provided.

16. List of international standards that the medical device complies with

EN ISO 13485; EN ISO 14971; EN ISO 14155-1; EN ISO 14155; EN ISO 10993-5; EN ISO 15223-1; EN 1041; MEDDEV 2.7/1 Rev. 4; MEDDEV 2.12/1 Rev. 8; EN ISO 17665-1; EN ISO 11138-1; EN ISO 11737-1; ISO 17665-1; ISO 11135-1; ISO 13397-2.

Note: The latest editions of the standards are used: the organization maintains and updates the list of reference standards as a record of its own management system in accordance with ISO 13485.

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LEE, JUNGYONG -----

attorney-in-fact of
DENTAL STUDIO CO., LTD.
CEO KIM BEOM JUN -----

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appeared before me and admitted
said principal's subscription to
the attached
INSTRUCTIONS FOR USE FOR THE
MEDICAL DEVICE. -----

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This is hereby attested on this
15th day of Jan. 2025 at this office.

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Belong to Seoul Central
District Prosecutor's Office
(Cheongjin-dong, Samsong Bldg.) 701, 702,
34, Jong-ro 3-gil, Jongno-gu, Seoul,
Republic of Korea

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Signature of the Notary Public
Lee Kyung Taek

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This office has been authorized by the
Minister of Justice, the Republic of
Korea, to act as Notary Public Since
7, Feb. 2020 Under Law No.15150.

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